

AVAILABLE THROUGH:




ACS, ALCOHOL COUNTERMEASURE SYSTEMS, ALCOLOCK and the "Molly" are trademarks of Alcohol Countermeasure Systems (International) Inc. and are used under license.



Alcohol Countermeasure Systems Corp  
60 International Boulevard  
Toronto, Ontario M9W 6J2 CANADA  
T +1 416 619 3500  
F +1 416 619 3501

20151030

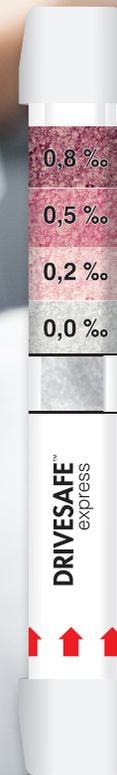

## DRIVESAFE™ express Single-Use Breathalyzer

DRIVESAFE™ express – a discrete, convenient, noninvasive, single use breath alcohol tester providing accurate, reliable results with easy to read color changing crystals. It's perfect for personal or professional use.

DRIVESAFE products have been trusted by transportation safety professionals for over 25 years.

# Single-Use Breath Alcohol Tester

## Trusted Service

DRIVESAFE breath alcohol testers have been trusted by professionals for 25 years for personal and workplace safety testing. With its ease of use and reliable performance, DRIVESAFE testers provide a single point of reference for an unbiased measurement of Breath Alcohol Concentration (BrAC).

## Convenience and Safety

The DRIVESAFE express is a hygienic, low cost, disposable, single use breath alcohol tester. It provides a quick and convenient way to measure your BrAC and can be carried in your pocket, purse or briefcase.

## Clear Results

	00	20	50	80
Canada (mg/dL)	00	20	50	80
Europe (mg/L)	0.00	0.10	0.25	0.35
Scandinavia (‰)	0,0	0,2	0,5	0,8
USA (%)	.00	.02	.05	.08

The DRIVESAFE express contains crystals within a single tube. When the user blows through the tube, the crystals change colour upon detection of alcohol; the more intense the colour, the higher the alcohol concentration. The DRIVESAFE express uses a four level indicator to easily see your BrAC within 2 to 4 minutes.

## Ease of Use


The DRIVESAFE express is easy to use with simple to understand instructions and no required training.

### Step by step


1. Firmly push both ends of the tube to break the security foil.



2. Blow for 10 seconds in the direction of the arrow.

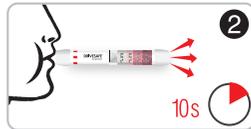

3. Wait 2 to 4 minutes and compare the colour of crystals to the printed scale to estimate your BrAC level.

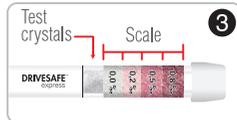

4. If you are over the limit, call a friend or cab for a safe ride home.

## Why use the DRIVESAFE express?

Unlike other methods that require blood or urine samples, the DRIVESAFE express is a noninvasive breath alcohol tester suitable for all environments.

For **professional use**, quickly measure an employee's readiness to work.

For **personal use**, its compact design makes it convenient for your purse or pocket, ideal for social or corporate gatherings.

